

# backRAP™



The backRAP™ is used for post-injury or post-surgery care. The backRAP™ is made from the finest quality of stretch absorbent material and provides gentle soft tissue support. It secures sterile dressings in place without the use of harsh medical tapes. It's unique design features a special pocket to hold the koolPAK™ in place for cold therapy/pain relief.

- Tapeless support of post operative dressing
- Eliminates tape blisters
- Easy access pocket for cold therapy
- Can reduce post-operative complications
- Decreases wound care time
- Latex-free
- Greatly improves patient comfort



• *koolPAK™ cold therapy placement*



• *Pocket access area between layers of fabric*



• *Velcro closures to adjust fit*

## INSTRUCTIONS

1. Place a sterile dressing over the surgical wound.
2. The backRAP™ is placed directly over the lower back and abdominal area. The wide portion with the pocket is placed over a sterile dressing on the low back or abdomen. The attachment is over the abdominal area in the front of the patient. See FIGURE 1 on opposite side.

3. Adjust the tension on the anterior attachment for support, but not so tight that it is constrictive or uncomfortable.
4. Smooth and adjust the backRAP™ so that there are no wrinkles and that the edges do not bunch. Adjust the wrap often to ensure comfort.

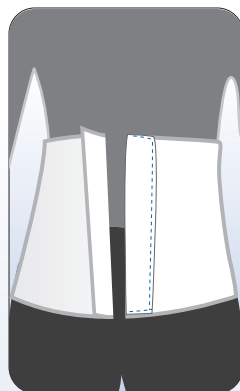


FIGURE 1

## PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a **MEDICAL PROFESSIONAL ONLY**.

The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are **NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND**. The sizing chart is approximate, the optimal size may need to be determined at the time of application. **DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.**

*For more information, please contact us  
and we will be happy to assist you.*



[www.sunmedica.com](http://www.sunmedica.com)

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715

1661 Zachi Way, Redding, CA 96003

# backRAP™

ORDER #	SIZE	WEIGHT
060-11-P/S	Petite/Small	90-140 LBS
060-11-M/L	Medium/Large	140-220 LBS