




hipRAP™ **EXTENSION**

ORDER # 003 - 11 - Ext.

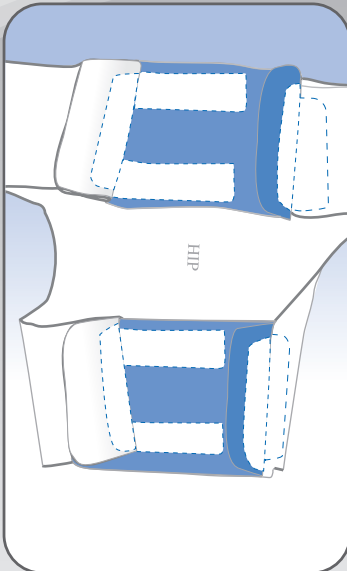

Waist & Thigh Extensions for the Difficult-to-Fit Body Shapes

The hipRAP™ Extension will allow a proper fit of the hipRAP™ on extra large patients.

- Eliminates tape blisters •
- Greatly improves patient comfort •
- Can reduce post-operative complications •
- Decreases wound care time •
- Latex-free •


INSTRUCTIONS

1. Extensions are to be used for the larger shaped bodies. They should be utilized for a better fit if the initial hipRAP™ does not fit correctly.
2. Attach the extension at the waist and/or thigh by using the hook and loop attachments as shown above. For application of the hipRAP™, please see hipRAP™ instructions within product box.

A *Stitching from opposite side*

B *Velcro/attachment area*

FIGURE 1

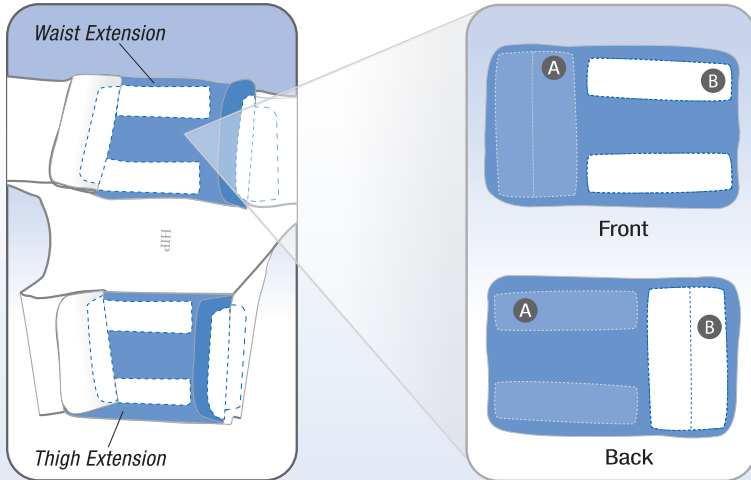

(The Waist and Thigh Extensions are identical)

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated.

These wraps should be applied properly by a **MEDICAL PROFESSIONAL ONLY**. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are **NOT STERILE**. **DO NOT APPLY DIRECTLY OVER WOUND**. The sizing chart is approximate, the optimal size may need to be determined at the time of application. **DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.**

For more information, please contact us and we will be happy to assist you.



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715
1661 Zachi Way, Redding, CA 96003