


hipRAP™



The hipRAP™ is used for post-operative, post-injury patient care. The hipRAP™ is made from the finest quality of stretch absorbent material. It provides soft tissue support and secures sterile dressings in place without the use of harsh medical tapes. It's unique design features a special pocket to hold the koolPAK™ in place for cold therapy/pain relief.

- Tapeless support of post-operative dressing
- Eliminates tape blisters
- Greatly improves patient comfort
- Can reduce post-operative complications
- Decreases wound care time
- Latex-free
- Easy access pocket for cold therapy

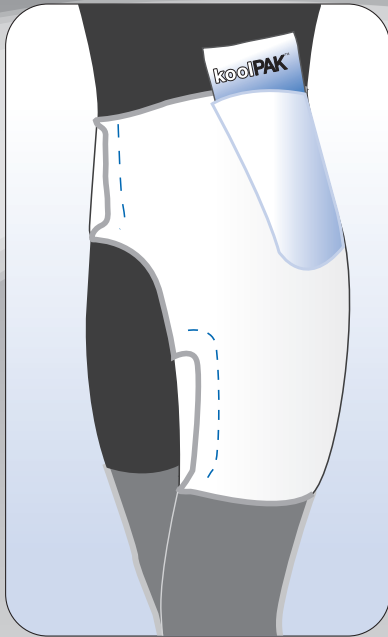

- *koolPAK™ cold therapy placement*



- *Pocket access area between layers of fabric*



- *Velcro closures to adjust fit*

INSTRUCTIONS

1. Place a sterile dressing over the wound. **DO NOT USE TAPE.**
2. The hipRAP™ is placed directly over the sterile dressing on the lateral waist-hip-thigh areas. You should be able to read "right" after it is applied to the right hip; "left" when applied to the left hip.
3. Adjust the hipRAP™ so the groin areas have ample space for peroneal care and body functions.
4. Slide the waist portion around the back remove the felt from the hook and attach **THE HOOK & LOOP ON THE ANTERIOR ABDOMEN** as shown in FIGURE 1.
5. Apply the thigh portion across the buttock, around the inner thigh and connect the hook and loop attachment. Use the natural contours of the patient to guide the position of the hook/loop attachments so the support is uniform.

6. Smooth and adjust the hipRAP™ so that there are no wrinkles and that the edges do not bunch. Adjust the wrap often to ensure comfort. For obese patients, see special application instructions.

SPECIAL APPLICATION INSTRUCTIONS FOR OBESE PATIENTS

1. The application of the hipRAP™ on obese patients needs special attention or irritation may occur from the wrap over the media-proximal thigh (inner thigh).
2. The thigh portion of the hipRAP™ should be applied to prevent tightness on the inner thigh. As per instruction of #4, the thigh hook and loop closure should be guided by the natural contour of the thigh. Specifically obese patients have accentuated conical shapes. The thigh closure should be angled to prevent tightness of the inner thigh portion of the wrap as shown in FIGURE 2.
3. *Extensions are available and should be used to allow appropriate application without causing tightness or irritation to the thigh.*

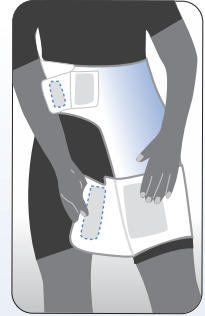


FIGURE 1



FIGURE 2

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a **MEDICAL PROFESSIONAL ONLY**. The patient should be frequently monitored for signs of discoloration or circulatory problems.

These products are **NOT STERILE**. **DO NOT APPLY DIRECTLY OVER WOUND**. The sizing chart is approximate, the optimal size may need to be determined at the time of application. **DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.**

*For more information, please contact us
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715

1661 Zachi Way, Redding, CA 96003

hipRAP™

HEIGHT	WEIGHT	41	45	50	54	59	64	68	73	77	82	86	91	95	100	104	109	113	KG	
CM	FEET	90	100	110	120	130	140	150	160	170	180	190	200	210	220	230	240	250	LBS	
147	4'10"	PETITE	PETITE	SMALL	SMALL	SMALL	MEDIUM	MEDIUM												
152	5'	PETITE	PETITE	SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM											
157	5'2"	PETITE	PETITE	SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE
163	5'4"	PETITE	PETITE	SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE
168	5'6"		PETITE	SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE
173	5'8"			SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE
178	5'10"				SMALL	SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE
183	6'					SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE
188	6'2"						SMALL	SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE
193	6'4"							SMALL	SMALL	MEDIUM	MEDIUM	MEDIUM	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE	LARGE

PETITE
 SMALL
 MEDIUM
 LARGE
 X-LARGE

This chart is approximate. Please see package insert for instructions.