

kneeRAP™



The kneeRAP™ is used for post-operative, post-injury patient care. The kneeRAP™ is made from the finest quality of stretch absorbent material. It provides soft tissue support and secures sterile dressings in place without the use of harsh medical tapes. It's unique design features a special pocket to hold the koolPAK™ in place for cold therapy/pain relief.

- Tapeless support of post operative dressing
- Eliminates tape blisters
- Greatly improves patient comfort
- Can reduce post-operative complications
- Decreases wound care time
- Latex-free
- Easy access pocket for cold therapy



• *koolPAK™ cold therapy placement*



• *Pocket access area between layers of fabric*

INSTRUCTIONS

1. The kneeRAP™ is rolled into a “donut” shape (FIGURE 1) starting from the large (proximal) end down to the bottom of the cold therapy pocket.
2. Pull the kneeRAP™ over the foot, ankle and lower leg until the bottom of the cold therapy pocket is just below the knee. The seam should be posterior.
3. Roll the kneeRAP™ over the knee and thigh. A dressing should be placed between the wrap and any incision or wounds.

4. Smooth and adjust the wrap so that there are no wrinkles or bunched edges. Adjust the wrap often to ensure comfort (FIGURE 2).

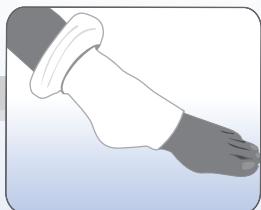


FIGURE 1

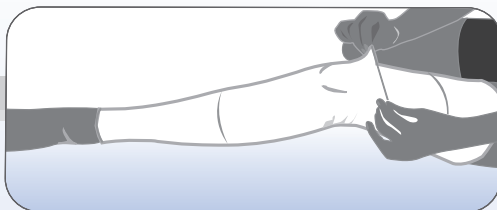


FIGURE 2

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a **MEDICAL PROFESSIONAL ONLY**. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are **NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND**. The sizing chart is approximate, the optimal size may need to be determined at the time of application. **DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.**

*For more information, please contact us
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715
1661 Zachi Way, Redding, CA 96003

kneeRAP™

ORDER #	SIZE	KNEE CIRCUMFERENCE
013-11-P	Petite	11"-13"
013-11-S	Small	13"-16"
013-11-M	Medium	16"-19"
013-11-L	Large	19"-21"
013-11-XL	X-Large	21"-23"